

Instructions for using Bi-Polar Kit With MM513 and Saddle Block (wear gloves)

- 1) Insert the Bi-Polar Fixture in the Saddle (MMSBT-170) in the slot for the LSE Ring Electrode



- 2) Remove the Clamp from the end of the Green Ground wire
- 3) Insert either end of the black connector adapter onto the end of the Green wire



- 4) Insert the Brush Electrode into the end of the black adapter
- 5) Insert the Bi-Polar instrument under test into the top of the Bi-Polar Fixture



- 6) Turn the MM513 unit on
- 7) Touch the bare tip of the Bi-Polar forceps to insure that the alarm sounds and the system is operational.
- 8) Using the Brush slowly brush each tine on the outside and inside
- 9) If the Alarm sounds the Bi-Polar has a fault, follow standard hospital procedures for a defective electro-surgical instrument.



www.mcgan.com

210 South Main St Unit 1B
Middletown, CT 06457
Ph: 508-876-1070 Fax: 508-883-3434